


Set Up Hardware

For devices with a Table Arm:




Locate the C-clamp (c), Allen key (a), and base plate (b). Locate a sturdy table edge and tighten the C-clamp (c) by using the provided Allen key (a). Be sure to attach the base plate (b) to the bolt before tightening the C-clamp down. Locate the arm (d). Slide the arm (d) into the C-clamp (c).

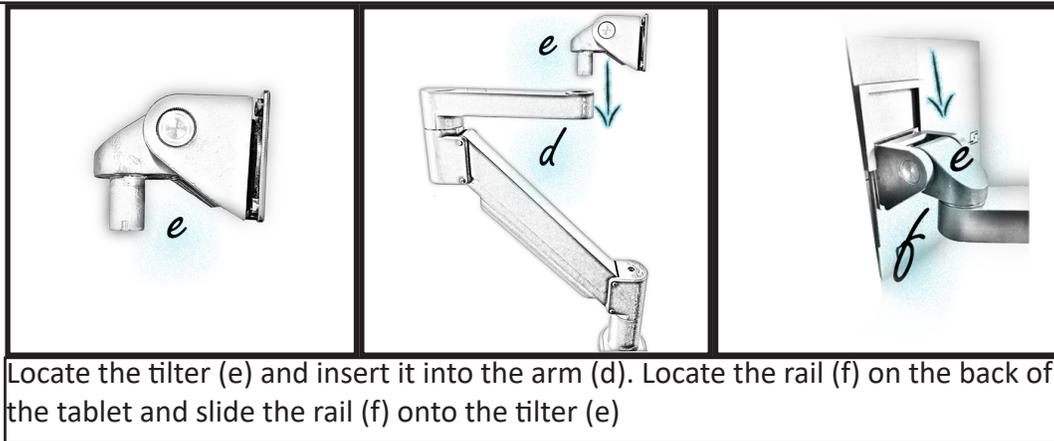

Locate the tilter (e) and insert it into the arm (d). Locate the rail (f) on the back of the tablet and slide the rail (f) onto the tilter (e)

For All Devices:

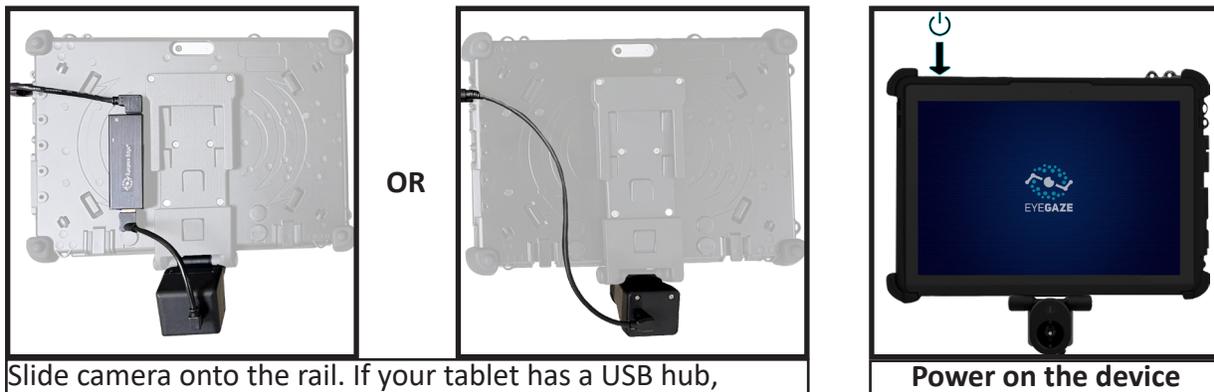

Slide camera onto the rail. If your tablet has a USB hub, connect the tablet to the hub, then the hub to the camera. If there isn't a hub, connect the tablet directly to the camera.

Power on the device